

EATA

How to wear

1



Open the package, take out the mask and open it.

2



Place the nose clip on the top of the mask and place the mask against the chin.

3



Pull the ear loop to the back of the ear and adjust the position as comfortable as possible.

4



Move and press from the middle of the nose clip to both sides, until the nose clip fits the nose.

5



Check the tightness of the mask and face.

Precautions

1. Please use as soon as possible after opening.
2. Do not use if you find outdated packaging, damaged or contaminated.
3. This product cannot be used as a gas mask.
4. From a health point of view, the recommended use time is 12 hours.
5. Mask needs to be replaced in time after contacting patients or suspected patients.
6. Should not be worn mask during sleep.
7. If you feel dizzy or uncomfortable, stop using it.
8. Patients with COPD, cardiovascular disease and pregnant women use this mask with caution.
9. In order to achieve better protection, please wear it in the correct way.

Product information

Product name: Ear loop disposable anti-bacteria dust mask

Model: EATA-KN9501L

Size: L (For adults)

Standard: GB2626-2006 KN95 (FFP 2)

Storage conditions: Dry and ventilated place without corrosive gas

Storage temperature: -30°C ~ 40°C, Storage humidity is less than 80%

Validity: 3 years

Production date: check on seal position

Manufacturer: ZDEATA Science Technology Co., Ltd

Company address: No. 10, Dakang Road, Henggang Industrial Zone, Shenzhen, China.

Made in China

